**medi USA**

6481 Franz Warner Parkway

P.O. Box 3000

Whitsett, NC 27377

**mediusa.com** | phone 1.800.633.6334 | fax 1.888.570.4554

medi

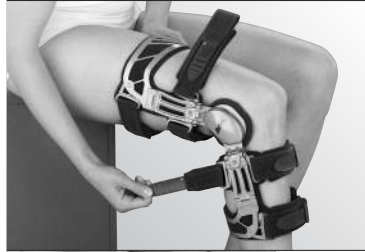
# M.4<sup>®</sup>s OA

## knee brace for varus/valgus relief

INSTRUCTIONS FOR USE / INSTRUCCIONES PARA USO

1

6

2

7

3

8

4

9

5

# M.4<sup>®</sup>s OA

*Dear Patient,  
We are pleased that you have decided to use the M.4<sup>®</sup>s OA and thank you for the trust you have shown in medi USA. Please read these instructions before using the orthosis for the first time. If you apply the orthosis correctly, it will be both therapeutic and comfortable to wear – so that you feel better.*

## Indications

- Moderate to severe unilateral arthrosis of the knee joint combined with ligament instability.
- Severe overloading of the medial or lateral compartment.
- Cartilage surgery/chondroplasty/meniscus refixation/transposition osteotomy with the necessary post-operative relief of the medial or lateral compartment.

## Side-Effects / Contraindications

Medial and lateral osteoarthritis in the same knee.

## Mode of Action

This orthosis is intended to help you to reduce stress on your knee. The orthosis is used in both postoperative therapy and in conservative (non-operative treatment). It protects the leg from swerving outwards. Stress is reduced on the painful areas of the knee joint and the orthosis brings a perceptible reduction in pain. Some patients only improve gradually.

## Setting the flexion and extension limits (only to be performed by an orthopaedic technician)

The orthosis is equipped with a 0° extension wedge. The orthosis includes additional wedges. The extension and flexion limits should only be limited in accordance with the instructions from the prescribing physician. To avoid hyperextension, extension wedges must be used. The wedges exactly specify the radius of motion of the orthosis. The condition of soft tissue may cause compression, making it necessary to use a larger wedge to restrict the movement of the knee to within the desired radius.

Select the required wedge. Extension wedges are marked with an E and flexion wedges with an F. Insert the wedge into the joint and lock it by clamping it with help of the frame ends. The wedge then slides automatically into the correct position and does not rotate. Tighten the wedge using the screw. Hand tightening will be sufficient.

## Setting the varus-valgus decompression

**Tip:** Get the responsible technician to give you a thorough explanation about how to adjust the M.4<sup>®</sup>s OA. This will help you to master the handling rapidly and to benefit from optimal support from the orthosis.

Sit down and adjust the varus-valgus adjustable joints to the neutral position, using an Allen wrench. You have reached the neutral position when the padding of the orthosis joint is covering the surface, but producing little or no pressure.

Stand up and set the tension to a quarter turn (on both adjusting screws). This is a good initial setting for the first time you wear the orthosis. If you want, you can adjust the orthosis to the breadth of your knee with the various joint pads enclosed. Take care that the orthosis is sitting properly.

For **conservative therapy** (non-surgical), the orthosis should not be worn for more than two hours per day. You can then gradually increase the hours of wear depending on how well you tolerate it. It is most important to wear the orthosis for activities that normally cause pain. By using the Allen wrench, the decompression can gradually be adjusted over the course of a day.

For **postoperative therapy**, the orthosis must always be worn when there is stress. Please ask your doctor for instructions.

## Caution:

Extreme settings have no therapeutic value, they only cause additional discomfort.

## Putting on / Taking off

- Open all the straps and pull the numbered end from the strap eyelets.

**Tip:** To prevent the strap from getting stuck, close the velcro tab in the attachment.

- Sit on the edge of a chair and spread the orthosis over the affected leg (**illustration 1**). Slightly bend your leg.
- Place the orthosis in such a position that the wedges visible from the front are level with the upper edge of the kneecap (**illustration 2**). Please take care that the orthosis is the right way round on your knee.
- Now close the lowest strap on the front side (strap 1) and the top strap at the back (strap 2) (**illustrations 3 and 4**). Press the joints gently backwards (**illustration 5**).
- Now close the lower strap at the back of the thigh (strap 3). Check that the strap is fitting properly. The lateral joints should be positioned slightly backwards (behind the lateral middle line of the knee).
- Now close the front upper lower leg strap (strap 4), followed by the back lower leg strap (strap 5) and finally the front thigh strap (strap 5) and then the front thigh strap (strap 6).
- Take a few steps or bend your knee a few times to check the fit. If necessary, the strap can be readjusted in the same sequence to guarantee a good fit.

If you suffer excessive pain or discomfort while wearing the brace, please remove the product immediately and contact your doctor or specialty store.

#### **What can I do if...**

##### **... the brace slips?**

- Check that the brace is fitted properly. The center of the brace joints should be level with the upper border of the kneecap.
- Make sure that the back thigh strap (#5) is above the belly of the calf muscle.
- Take the covers off the knee pads and wear the pads directly on your skin. Watch carefully for skin reactions. Skin irritation may occur in people with very sensitive skin.
- Do not wear the brace over clothes.

##### **... the pads have worn down?**

- Replacement pads can be obtained from medi USA. Your orthotist will be pleased to help.

---

#### **Care instructions**

The inner pads and the condyle pads can be washed by hand with a mild soap/detergent. Leave to dry naturally. When used normally the brace is maintenance-free. However, after using the brace in water, the hinges can be lubricated using Teflon™ spray. Check regularly that the joint screws are secure, tighten them if necessary.

---

#### **Storage**

Must be stored dry and at room temperature.  
Must be protected from the sun and moisture.

#### **Statutory Defects Liability / Contractual Guarantee**

Consistent with legal guidelines.

#### **Important Notes**

The orthosis is used for a single patient only. If the orthosis is used for the treatment of more than one patient or is improperly modified, the manufacturer will not be liable.

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