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AC Abduction Pillow

IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct application is required for proper function. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INDICATIONS

- Rotator cuff repairs
- Glenohumeral dislocations/subluxation
- Capsular shifts
- Bankart repairs
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

CAUTIONS

This product is to be fitted by a properly licensed practitioner who is familiar with the purpose for which it is intended. The practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your practitioner immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your practitioner for the duration of use.

INSTRUCTIONS FOR USE

NOTE: Ensure all open wounds are covered before applying device

APPLYING THE DEVICE

1. Brace is initially configured to fit the patient's right shoulder. To configure for the left shoulder, release both buckles at the forearm cuff. Rotate the sling and shoulder strap 180 degrees and reattach both buckles. **NOTE:** Ensure the front shoulder strap is not twisted before reattaching the buckles.
2. Release the front buckle clasp on the shoulder strap where the strap attaches to forearm cuff on sling. Remove the thumb and forearm straps to completely open the sling.
3. Place affected arm in sling so it rests comfortably in the fold of the sling. Ensure patient's elbow is seated as far back in the fold pocket of the sling as possible. (fig.1)

4. The sling should extend to the first joint of the pinky finger. Sling length can be adjusted by removing the forearm end of the cuff and re-attaching it in the desired position. **NOTE:** Attach forearm cuff towards elbow pocket to make sling shorter and away from the elbow pocket to make longer.
5. Reattach the front buckles on the shoulder strap to the ends on the forearm cuff of the sling. (fig. 2) Reattach the forearm strap on sling.
6. The thumb loop can now be reattached as needed per instructions of the practitioner. (fig. 3)
7. Shoulder strap length can be adjusted at both ends of the straps and at the shoulder rings for added comfort and support. To adjust the shoulder straps, simply release the hook strap and reattach at the desired length. (fig.4) **NOTE:** If extensive shortening of the straps is necessary, the Y-tabs at the end of the straps can be removed so the strap can be trimmed to the desired length.
8. The foam block is designed with the buckle end on the front for ease of use. Detach the buckle at end of the foam block. Place waist strap around patient's waist and fasten buckle. (fig. 5) To adjust strap length, pull the waist strap for a secure fit. The position of the foam block may be adjusted by sliding the block forward or backward along the waistline to achieve the desired external or internal rotation. **NOTE:** If strap is too long, remove Y-tab and trim strap to desired length.
9. Align the foam block at waistline on patient's affected side. Attach arm sling to the foam block by securing hook and loop portions together. (fig. 6)

REMOVING THE DEVICE

1. Open the forearm Velcro strap and the thumb loop strap
2. Disconnect the waist strap buckle
3. Disconnect the two shoulder strap buckles and pull the shoulder strap off.

CARE AND WASHING INSTRUCTIONS

1. Store in a cool, dry place
2. Hand wash only

MATERIAL COMPOSITION

- Nylon
- Polyester
- Polyurethane